
Disposable Reservoirs

Application Scope

Shall be used with 3000 series ambulatory infusion pump produced by Medsure Medical Technology Co., Ltd, the reservoirs is for single use.

Filling medicine solution

- Unpack, take out the reservoirs, remove the luer protective cap, and release the flow clip A.
- Inject the medicine solution into the medicine bag through the female luer B with a syringe.
- Shake the reservoirs in different planes so that all air gathers at the outlet of the medicine bag and use a syringe to draw out the air until there aren't bubbles in the bag and tubing. While drawing out air, the reservoirs is placed vertically as shown in the figure.
- The injected medicine solution shall not exceed the nominal volume of reservoirs, otherwise it may be damaged.
- Remove the anti-leakage clip insert piece C, assemble the reservoirs to the 3000 series ambulatory infusion pump, lock it, connect the extension tubing through the female luer (pay attention to the connecting direction of the extension tubing), loose the flow clip A, then start infusion.

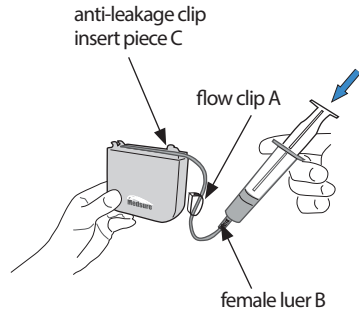Cautions

- The reservoirs is sterile and disposable.
- The reservoirs is sterilized by ethylene oxide and its period of validity is three years.
- Please check the package and components before use. If the product is damaged or has no luer protective cap, please do not use it.
- The application environment temperature of reservoirs is $+5\text{ }^{\circ}\text{C} \sim +40\text{ }^{\circ}\text{C}$, and the time for infusion shall be less than 72 hours.
- Please don't overfill or over-squeeze the reservoirs. The internal pressure of the reservoirs after filling the nominal volume shall be less than 20 kPa.
- The operation of reservoirs shall meet the requirement of the relevant medical department operational specifications and relevant regulations, and only for trained professionals.
- The material of reservoirs is mainly polyvinyl chloride. The clinical staff need pay attention to the compatibility of infused drugs and reservoirs. It is forbidden to infuse drugs that are incompatible with polyvinyl chloride.
- The maximum pressure that can be generated by the 3000 series ambulatory infusion pump does not exceed 180 kPa.
- Clinical staff shall pay attention to the pharmacodynamic changes due to the interaction of the PVC tubing and the infused drug.

Transport and storage conditions

- The reservoirs shall be stored in a dry, non-corrosive gas and well-ventilated room.
- Do not throw the reservoirs during loading and unloading, avoid strong vibration during transportation, prevent heavy pressure, direct sunlight, rain and snow.

Contraindications

- The reservoirs contains DEHP. " The reservoirs shall not be used to store and infuse the fat-soluble liquids or drugs such as fat emulsion (such as propofol, paclitaxel, etc.)". "Newborn, prepubertal male, pregnant and lactating women are not recommended to use the reservoirs to infuse drugs".

Warning

- The 3000 series ambulatory infusion pump has a flow accuracy error. In order to ensure that the medicine solution in the bag is not infused completely before the expected time, the error value shall be considered when filling the reservoirs.

Symbol Meaning

Batch code

Use-by date

Do not re-use


STERILE EO

Sterilized using ethylene oxide

Product structure

The disposable reservoirs is composed of casing (1), medicine bag (2), baffle (3), elastic PVC tubing (4), anti-leakage clip insert piece (5), anti-leakage clip (6), spring (7), flow clip (8), tubing (9), female luer (10), extension tubing (11), flow filter (12), anti-backflow valve (13), luer protective cap (14).

Components contacting with medicine solution are medicine bag, elastic PVC tubing, tube, female luer, extension tubing, flow filter and anti-backflow valve. The adhesive for the tubing and the elastic PVC tubing is cyclohexanone. The plasticizer in PVC is DEHP.

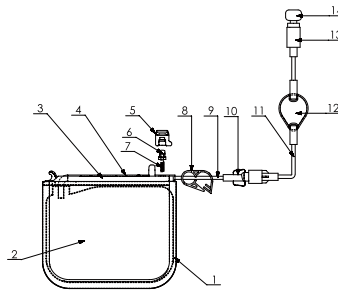Specification

Product Category	Product Model	Nominal Volume/mL	Color
Disposable Reservoirs	C3-50	50	Blue
Disposable Reservoirs	C3-100	100	Blue
Disposable Reservoirs	C3-150	150	Blue
Disposable Reservoirs	C3-200	200	Blue
Disposable Reservoirs	C3-300	300	Blue

Registrant/Manufacturer: Medsure Medical Technology Co., Ltd.

Residence address: Building 2, No.8 Keling Road, Suzhou New District, 215163 Suzhou, PEOPLE'S REPUBLIC OF CHINA

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Kochstr. 1, 47877, Willich, Germany

Production/After-Sales address: Building 2, No.8 Keling Road, Suzhou New District, 215163 Suzhou, PEOPLE'S REPUBLIC OF CHINA

Contact

Tel./Fax: +86-512-66919880/66916865

Postal Code: 215163

After-sales service: Medsure Medical Technology Co., Ltd.

Batch Code: See minimum package for details

Date of Manufacture: See minimum package for details

Use-by date: See minimum package for details

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Luxus Lebenswelt GmbH